

# Gutta-percha points for canals

## INSTRUCTIONS FOR USE

Gutta-percha is one of the dental restorative materials. Dentists use it as a root canal filler when treating endodontic diseases. Our gutta-percha got the ShangHai New Excellence Product award.

1. It is white (or pink) spicule, odorless, without irritation with disinfection.
2. It is impervious to X-rays and observe dental root-canal filling clearly.
3. It has certain tenacity. It can easily get into the fine and obstructed root canals.
4. Its diameter and size are in accordance with root canal reamers.

## MAIN INGREDIENTS

High quality natural gutta-percha, zinc oxide.

### 1) PURPOSE

Used in root-canal for obturating points when dentist treats endodontic disease.

### 2) CONTRAINDICATIONS

None known.

### 3) WARNINGS

- It should be disinfected with alcohol before use.
- It Should not be applied to allergic patients.

### 4) SPECIFICATIONS

- It is produced in accordance with ISO 13485. It can be customized in size according to customer's requirements, with a variety of sizes.

| Zarc Gutta-percha size 0.04 and 0.06 |        |        |        |        |
|--------------------------------------|--------|--------|--------|--------|
| Standard specification               | .04/25 | .04/30 | .06/25 | .06/30 |
| Length (mm)                          | 28     | 28     | 28     | 28     |
| Diameter tip (mm)                    | 0.25   | 0.30   | 0.25   | 0.30   |

| Zarc Gutta-percha special size |        |      |
|--------------------------------|--------|------|
| Denomination                   | X-Fine | Fine |
| Length (mm)                    | 28     | 28   |
| Diameter tip (mm)              | 0.14   | 0.17 |

| BlueShaper® gutta-percha size |      |      |      |      |      |
|-------------------------------|------|------|------|------|------|
| Denomination                  | Z3   | Z4   | Z5   | Z6   | Z7   |
| Length (mm)                   | 28   | 28   | 28   | 28   | 28   |
| Diameter tip (mm)             | 0.20 | 0.25 | 0.30 | 0.40 | 0.50 |

| Excalibur® gutta-percha size |      |      |      |      |
|------------------------------|------|------|------|------|
| Denomination                 | E20  | E25  | E35  | E45  |
| Length (mm)                  | 28   | 28   | 28   | 28   |
| Diameter tip (mm)            | 0.20 | 0.25 | 0.35 | 0.45 |

| SlimShaper® gutta-percha size |      |      |      |
|-------------------------------|------|------|------|
| Denomination                  | ZS1  | ZS2  | ZS3  |
| Length (mm)                   | 28   | 28   | 28   |
| Diameter tip (mm)             | 0.15 | 0.20 | 0.25 |

| ApicalShaper® gutta-percha size |     |     |     |     |
|---------------------------------|-----|-----|-----|-----|
| Denomination                    | Z30 | Z35 | Z40 | Z50 |
| Length (mm)                     | 28  | 28  | 28  | 28  |
| Diameter tip (mm)               | 30  | 35  | 40  | 45  |

## 5) STORAGE AND EXPIRY

- Store in a cool place. Keep away from direct sunlight, irritating gases and smoke.
- Store as required.
- The shelf life of gutta-percha is 3 years.

## 6) PACKAGING

- Box package: 100 pieces/box or 60 pieces/box according to gutta-percha size. The label and signature with proof of conformity on the box.



Shanghai Xing Yu Medical Equipment Co., Ltd  
101 Room, 1st Floor, Building 6, No. 868  
Xin Ge Road, Song Jiang District,  
201612 Shanghai. PR. China  
hola@zarc4endo.com • www.zarc4endo.com



Luxus Lebenswelt  
GmbH Kochstr. 1  
47877 Willich,  
Germany



Omologic, Homologación y Marcado CE,  
S.L.U. Calle Santa Rosalía, 21,  
Escalera 2, 1ºB 18007 Granada  
(Spain)

IFU-Ver. 01

Gutta-percha points  
for canals

FOR DENTAL USE ONLY

