

# Absorbent Paper Points

## INSTRUCTIONS FOR USE

1. Paper Points are a white spicule and have definite tenacity and intensity. It can easily get into exiguous and bend root-canal.
2. It can strongly sop up tissue saliva and make it dry quickly.
3. It can keep straight when dipped in antimicrobial or disinfectant solutions for treating endodontic disease.
4. Its diameter and sizes are in accordance with root canal needles.
5. **STERILE:** It will be irradiated with 25 y-rays after packing.

## COMPOSITION

Cotton paper.

## 1) PURPOSE

Use in assimilating the oral cavity root canal tissue fluid when dentists cure endodontic disease.

## 2) CONTRAINDICATIONS

None known.

## 3) SPECIFICATION

| Zarc Paper Point size 0.04 and 0.06 |        |        |        |        |
|-------------------------------------|--------|--------|--------|--------|
| Standard specification              | .04/25 | .04/30 | .06/25 | .06/30 |
| Length (mm)                         | 28     | 28     | 28     | 28     |
| Diameter tip (mm)                   | 0.25   | 0.30   | 0.25   | 0.30   |

| Zarc Paper Point special size |        |      |
|-------------------------------|--------|------|
| Denomination                  | X-Fine | Fine |
| Length (mm)                   | 28     | 28   |
| Diameter tip (mm)             | 0.14   | 0.17 |

| BlueShaper® Paper Point size |      |      |      |      |      |
|------------------------------|------|------|------|------|------|
| Denomination                 | Z3   | Z4   | Z5   | Z6   | Z7   |
| Length (mm)                  | 28   | 28   | 28   | 28   | 28   |
| Diameter tip (mm)            | 0.20 | 0.25 | 0.30 | 0.40 | 0.50 |

| Excallbur® Paper Point size |      |      |      |      |
|-----------------------------|------|------|------|------|
| Denomination                | E20  | E25  | E35  | E45  |
| Length (mm)                 | 28   | 28   | 28   | 28   |
| Diameter tip (mm)           | 0.20 | 0.25 | 0.35 | 0.45 |

| SlimShaper® Paper Point size |      |      |      |
|------------------------------|------|------|------|
| Denomination                 | ZS1  | ZS2  | ZS3  |
| Length (mm)                  | 28   | 28   | 28   |
| Diameter tip (mm)            | 0.15 | 0.20 | 0.25 |

| ApicalShaper® Paper Point size |     |     |     |     |
|--------------------------------|-----|-----|-----|-----|
| Denomination                   | Z30 | Z35 | Z40 | Z50 |
| Length (mm)                    | 28  | 28  | 28  | 28  |
| Diameter tip (mm)              | 30  | 35  | 40  | 45  |

## 5) STORAGE

- Keep below 40°C, airless, in a non-windy and non-corrosive place. The maximum accumulation height is 5 boxes.

## 6) PACKAGE

- Box package: 100 pieces/box or 180 pieces/box according to paper point size. The label and signature with proof of conformity are shown on the box.

## 7) SPECIAL NOTE

- Please stop using it when the color changed (the normal color is white).
- If the innermost pipe is broken, don't use it.
- Single use only.
- Please use it in time after the package has been opened.

## 8) STERILIZATION

- Store according to request, three years of sterilization term.

## 9) INSTRUCTIONS FOR USE

- For dentist use only. Choose the same size as dental files and user tweezers to clip the paper point for cleaning the root canal. The products are for disposable use only.



 Shanghai Xing Yu Medical Equipment Co., Ltd  
101 Room, 1st Floor, Building 6, No. 868  
Xin Ge Road, Song Jiang District,  
201612 Shanghai. P.R. China  
hola@zarc4endo.com • www.zarc4endo.com

 Luxus Lebenswelt  
GmbH Kochstr. 1  
47877 Willich,  
Germany



Omologic, Homologación y Mercado CE,  
S.L.U. Calle Santa Rosalía, 21,  
Escalera 2, 1ºB 18007 Granada  
(Spain)